

FlexiFit™

S E R I E S

HC405
NASAL MASK

LAB REPROCESSING INSTRUCTIONS

185042996 REV B

Fisher & Paykel
HEALTHCARE

FISHER & PAYKEL HEALTHCARE

INTERNATIONAL	PO Box 14 348, Panmure, Auckland 6, New Zealand Tel: +64 9 574 0100 Fax: +64 9 574 0158 Email: info@fphcare.com Website: www.fphcare.com
USA	Tel: 1800 446 3908 or +1 949 470 3900 Fax: +1 949 470 3933
AUSTRALIA	Tel: +61 3 9879 5022 Fax: +61 3 9879 5232
AUSTRIA	Tel: 0800 29 31 23 Fax: 0800 29 31 22
BENELUX	Tel: +31 78 644 0924 Fax: +31 78 644 0914
FRANCE	Tel: +33 1 6446 5201 Fax: +33 1 6446 5221
GERMANY	Tel: +49 7182 93777 0 Fax: +49 7182 93777 99
IRISH REPUBLIC	Tel: +1800 409 011
ITALY	Tel: +39 06 7839 2939 Fax: +39 06 7814 7709
SPAIN	Tel: +34 902 013 346 Fax: +34 902 013 379
UK (EU Authorised Representative)	Tel: +44 1628 626 136 Fax: +44 1628 626 146
NTHN IRELAND	Tel: 0800 132 189

Fisher & Paykel
HEALTHCARE

FLEXIFIT™ SERIES - HC405 LAB REPROCESSING INSTRUCTIONS

Your HC405 nasal mask is provided patient ready. To re-use between patients please follow the guidelines below.

ASSEMBLY/DISASSEMBLY

- ① Place the Foam Cushion (D) into the Mask Base (A).
- ② Starting at the apex of the mask, fit the Silicone Seal (C) over the Foam Cushion and onto the Mask Base, finishing at the bottom of the mask.
- ③ Push the Elbow (B) into the Mask Base (A).
- ④ Lay the Headgear (K) flat with the loop side down. Position the assembled mask over the Headgear and attach the four straps into the corresponding slots in the Forehead Rest (I) and the Glider™ Strap (E). This can be done without undoing the Velcro™ tabs, by sliding the Headgear into the slots.









CLEANING / DESOILING

- Before disinfection, remove visible soil by disassembling and thoroughly cleaning the mask with a small brush under running potable tap water.

FAILURE TO PROPERLY CLEAN THE DEVICE MAY LEAD TO INADEQUATE RESULTS.

DISINFECTION PARAMETERS

- NB: For autoclave cycles, components should be enclosed in a sterilization peel pouch.

	AUTOCLAVING <small>Gravity displacement type autoclave, 15min exposure at 121°C (250°F).</small>	SPOROX II™ <small>Follow manufacturer instructions for high level disinfection. Do not dilute. Immerse for 6 hours at 20 °C. Rinse thoroughly in potable water.</small>
MASK FRAME 	YES	YES
SILICONE SEAL 	YES	YES
ELBOW 	YES	YES
FEMALE SWIVEL 	YES	YES
HEADGEAR 	<i>Wash in warm soapy water and leave to dry</i>	
FOAM CUSHION 	<i>Replace with new part</i>	
DIFFUSER 	<i>If continuing use with the diffuser, replace with a new part</i>	
DIFFUSER COVER 	<i>If continuing use with the diffuser, replace with a new part</i>	

THE HC405 NASAL MASK HAS BEEN VALIDATED TO WITHSTAND 20 CYCLES OF THE ABOVE PARAMETERS. DO NOT USE IF BADLY DAMAGED.

Disclaimer: It is the users responsibility to qualify any deviations from the recommended method of reprocessing. Contact your local Homecare Provider for additional information.

