

How to fit your **HC405** Nasal Mask

- 1** Place headgear over head and re-attach strapping to the glider. Pull the lower straps as low as possible down back of neck. Gently tighten straps to remove slack.
- 2** Attach your humidifier (HC150) or humidified CPAP (HC220 series) and turn on.
- 3** Adjust lower straps to eliminate mask leaks.
- 4** Check you are using the correct size seal. If you have pressure on your nose, you may need the medium/large silicone seal. If you experience leaks around the nose you may need the medium/small silicone seal.

